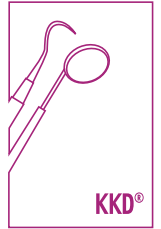
# KKD<sup>®</sup> *mf* U-CORDPACKER & U-FIXPLIERS

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*Universal Instrument for atraumatic insertion of all retraction cords into the gingival pockets in order to displace the marginal gingiva for the exposure of the preparation limit before impression taking and for the desiccation of the cavity in the Filling Therapy.*

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*U-Cordpacker is an instrument where an easy and atraumatic cord placing is possible, even with a thin and atrophy marginal gingiva.*

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*The principle of using the instrument is a threading working method, which is considerably gentler to the periodontal apparatus and clearly more comfortable for the patient.*

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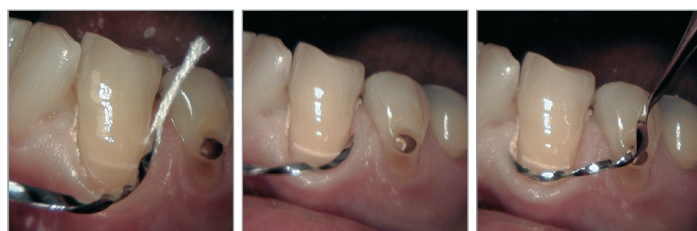
*With the threading working method the strength transmission occurs controlled, this way a force procedure is inhibited and an unnecessary traumatising of the periodontal apparatus is avoided.*

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
*The cord cannot slip away due to the torque of the working ends. The working ends are made out of a feather hard alloy that does not lose its feather quality after repeated sterilisation of the instrument.*

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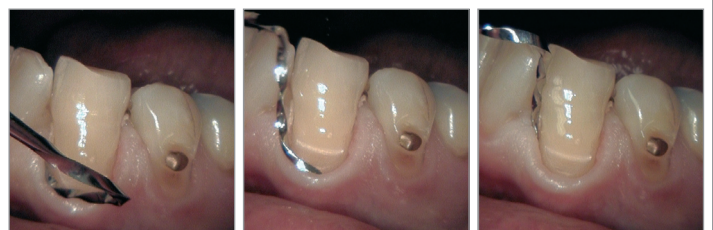*The crucial advantages of this instrument lie in its tooth anatomy and periodontal apparatus adjusted form of the working ends.*

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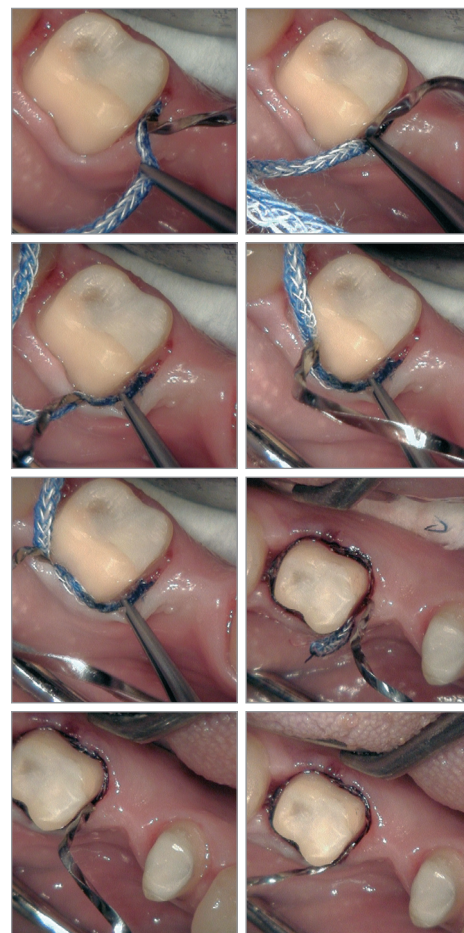
*The U-Cordpacker is a double-ended instrument where all points of the working ends are formed and placed opposite to each other. This way it is possible to make the same application in all areas around the tooth inversely. Even though the instrument is bent complicatedly, it suits both right and left handed persons. Due to the manifold bending and dimensioning of the working ends, it is possible to place the cord around the tooth under view.*

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*We recommend a bimanual placing of the retraction cord. The retraction cord is placed on the sulcus with the especially developed U-Fixpliers and then threaded in the sulcus with the U-Cordpacker. Afterwards the retraction cord is released from the pliers. When closed the gripping side pieces of the pliers look like a thin cement filling plugger. This way the retraction cord is held at the beginning of the sulcus and cannot slide out during the threading with the U-Cordpacker.*

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*The bimanual working method is gentler to the periodontal apparatus and clearly more comfortable for the patient because the strength transmission occurs controlled. The risk of an unnecessary traumatising of the periodontal apparatus is considerably lesser. The U-Fixpliers is a very well suited instrument for the bimanual working method. With a synchronal appliance, the special characteristics of the Cordpacker and the pliers allow a fast and efficient working method.*

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*The design of the U-Fixpliers allows to place and hold the retraction cord around the tooth under view. The bending of the side pieces are chosen so that it is possible to reach all regions in the oral cavity even when only slightly opened. The points of the pliers are dimensioned and formed so that they can fulfil two functions:*

- clamping and carrying
- punctual fixing of the retraction cord in the sulcus.

**U-Fixpliers**  
REF 13710

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