# KKD<sup>®</sup> mf U-CUTTER

## Abdruckschneidemesser

KKD<sup>®</sup>

### 01 KKD<sup>®</sup> mf U-Cutter nach Dr. M Ugrinovic

U-Cutter  
REF 12200

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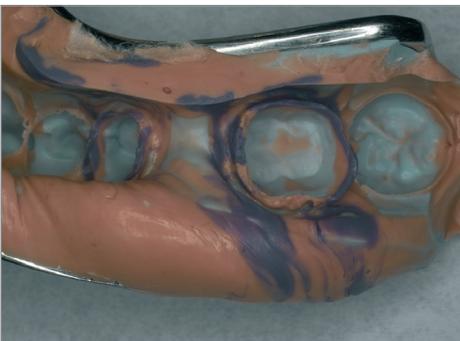
*Universal instrument for working on all common rubber-elastic impression materials, as A-silicones, K-silicones, polyether, alginates etc. The U-Cutter is a further development of the K-Cutter approved for years. The two working ends of the U-Cutter are bent several times at well-defined angles and have along the entire working area sharp edges on both sides. The corrective impression, inaugurated in 1955 by HOFMANN, is nowadays due to its practicability and best results, one of the most applied impression taking methods for fixed tooth replacement.*

*The principle of the technique is to correct the preliminary impression, made with a high-viscous impression material onto the prepared teeth, in a second operation with a low-viscous impression material. As a result of the bad flow off conditions, because of the geometry and mainly because of the relatively high precision of the preliminary impression, sources of error occur which can affect unfavourably the impression results.*

*With a system of flow off channels in the preliminary impression, by UGRINOVIC, a sufficient pressure during the impression taking is exerted which presses the low-viscous corrective material into the sulcus opened due to the retraction cord and, on the other hand, allows a flowing off over the soft parts.*

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*An atraumatic forming of the crown margins in periodontal-hygienic respects is only possible after an exact reproduction of the preparation limits. This is guaranteed by the corrective impression modified with the flow off channels even under most difficult clinical conditions. With the U-Cutter flow off channels can be achieved well-directed, exactly and very fast.*

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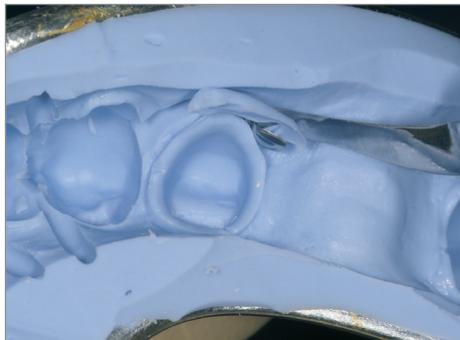
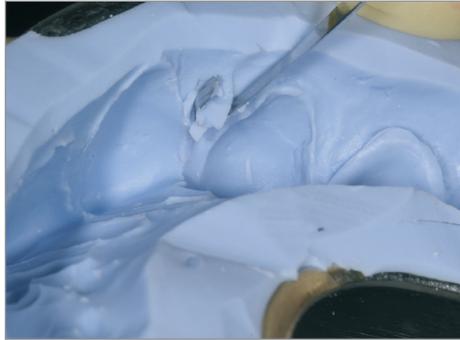
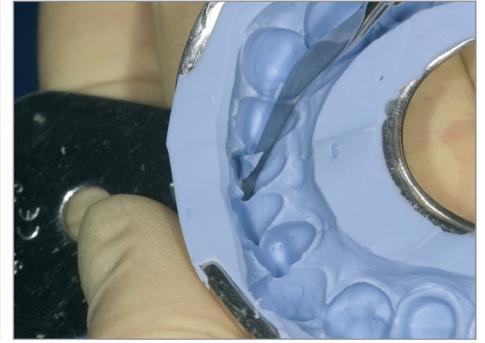
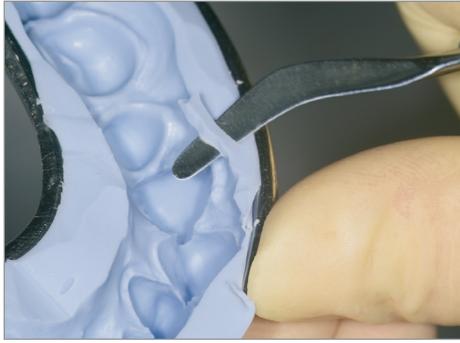
*The form of the cutting edges and their cutting capability on both sides permit to cut in two different directions. When cutting the impression material, the working ends slide over the rim of the tray, without cutting into the impression tray.*

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*With the gracile cutting edges of the U-Cutter also the sharp edges in the preliminary impression are removed. Even the deepest and narrowest impression areas can be reached without any difficulty.*

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
*The flow off channels are prepared consequently in the entire preliminary impression, starting from the middle of the approximal areas towards outside, both oral and vestibular. The channels just run in the region of the soft parts.*

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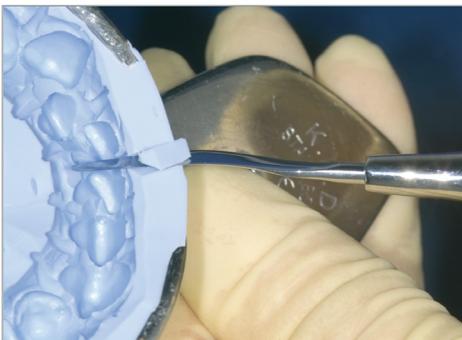*By preparing the marginal channel close to the prepared stump (also only in the region of the soft parts) the local flow conditions are changed so that stowages and a recirculation of the corrective material can be produced where desired.*

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*An especially ergonomic method of working is achieved due to the fact that you need just one instrument for the complete preparation of the preliminary impression. With the U-Cutter the entire preparation of the preliminary impression will take only 2 minutes!*

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