KKD[®] *mf* U-PROVI-CLEANER

Reinigungsinstrument

KKD[®]

01 KKD[®] *mf* U-Provi-Cleaner nach Dr. M. Ugrinovic

U-Provi-Cleaner
REF 13711

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For a fast and efficient cleaning of all provisional and final crowns and bridges from all common provisional cements. So far, Heidemann- and Hollenback spatulas or even some types of excavators have been used but without resulting in a fast and efficient cleaning. Also, these instruments are deformed and no longer useable for their proper application. That is the reason why the U-PROVI-CLEANER has been developed by Dr. Ugrinovic.

In general, crowns and bridges - prior to the final insertion - are inserted with a temporary fixing material (provisional cement) for the so-called „test-wearing“. Since crowns and bridges are very precisely fitting and gracile workpieces, the fit can be negatively influenced by remaining cement particles during final insertion. That is the reason why an entire removal of the provisional cement is absolutely necessary.

In general provisional crowns and bridges are made of materials which are relatively soft and brittle. Since provisional cements are tough and sticky, an efficient removal of the cement without a suitable instrument is only possible when exerting a certain pressure. Due to this pressure very often both the provisional and the final crowns are damaged.

Especially for final tooth replacement, damaging the crown margins and spalling off of the ceramics is very annoying and cost-intensive.

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The two nearly identical, but variably dimensioned forms of the working tips allow the use of the instrument in each crown lumen and inlay form.

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Instruction:

We recommend the following procedure with the U-PROVI-CLEANER for final and provisional restorations: during the cleaning procedure please sit down. Lay the forearms down on the working table. Do not hold the workpiece too high above the worktop. Put a plastic pad or towel on the worktop. These advices are mainly important when cleaning ceramic veneered tooth replacements. If you drop the workpiece, it falls from not too high on a soft ground. Final crowns and bridges are always high-shine polished and therefore very slippery.

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Hold the workpiece in such a way that the crown margins are supported by the finger tips. In that way the pressure exerted on the crown walls and especially on the crown margins is better controlled. Use the instrument in the crown lumen in such a way that the cement removal is carried out in a peeling off way exerting only a slight pressure.

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For temporary inlays the procedure is the same as described above

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The instrument is also suitable for the cleaning of full ceramic works

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